



## PRODUCT INFORMATION

**Novel synthetic guided wound closure matrix for difficult to heal wounds**

## PRODUCT SUMMARY

- **SUPRA SDRM®** is a novel synthetic, guided wound closure matrix, built as a bimodal foam membrane structure for the management of chronic wounds.
- **SUPRA SDRM®** combines the benefits of the microporous structure known from Suprathel® that builds a protective barrier allowing the wound to heal and additional large pores which were created to enable ingrowth of blood vessels.
- **SUPRA SDRM®** is fully synthetic – no animal derived components – polylactic acid as the main component reducing the biologic risk and inflammatory reactions.
- No preparation of **SUPRA SDRM®** matrix necessary.
- When applied to the wound **SUPRA SDRM®** adheres well to the wound bed without the need for direct fixation.
- **SUPRA SDRM®** becomes translucent after the application allowing healthcare professionals to easily assess the wound healing.
- **SUPRA SDRM®** is available in multiple sizes and can be trimmed to meet the patient's needs.


## INDICATIONS

**SUPRA SDRM®** is indicated for use in the management of:

- Pressure (stage I and IV) and venous Ulcers
- Ulcers caused by mixed vascular Etiologies
- Venous stasis and diabetic ulcers
- Partial and full thickness wounds
- Partial thickness burns
- Trauma wounds
- Surgical wounds

## SUPRA SDRM® APPLICATION QUICK GUIDE

- Apply **SUPRA SDRM®** and cover with non-adherent mesh dressing; apply outer dressings for desired drainage management to maintain a moist wound healing environment.
- SUPRA SDRM®** may be reapplied every 7 days or as needed until wound healing is achieved.
- Outer dressings may be changed if soiled.

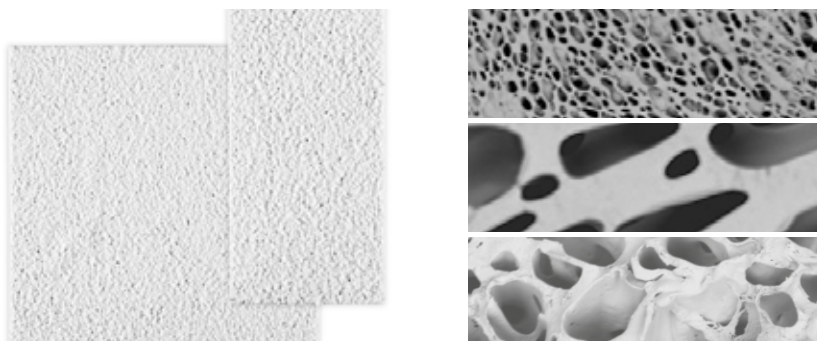

## PROPERTIES

Composition:	Lactide-based copolymer
Degradation	Hydrolytical (to CO <sub>2</sub> and H <sub>2</sub> O)
Thickness	1.5 - 2.1 mm
Porosity	> 85%, micro-pores 13 - 300 µm
Sterilization	Gamma Sterilization

## PACKAGING INFORMATION

Size	Units	Order-No.
1 x 1 cm	1	310101-US
18 mm disc	1	31D018-US
2 x 2 cm	1	310202-US
5.1 x 5.1 cm	1	310505-US
9 x 9 cm	1	310909-US
9 x 12 cm	1	310912-US
18 x 9 cm	1	311809-US
18 x 18 cm	1	311818-US

### Wound bed preparation



Debride devitalized tissue and refresh the wound with cross hatching as needed. Bring to hemostasis.

### SUPRA SDRM® application



Apply **SUPRA SDRM®** to cover wound surface.

### Non-adherent secondary dressing



Apply non-adherent dressing to keep **SUPRA SDRM®** in place. Apply outer dressings for desired drainage management.

For further information please contact

**PolyMedics Innovations Inc.**

8681 Highway 92, Suite 308, Woodstock, GA 30189

Phone +1 646-604-2771 | Fax +1 646-350-3129 | E-mail info.usa@polymedics.com



[www.suprasdrm.com](http://www.suprasdrm.com)