



PRODUCT INFORMATION

An effective guided wound closure matrix

PRODUCT SUMMARY

- **SUPRA SDRM[®] CW** is a biodegradable, guided wound closure matrix for the management of chronic wounds and difficult to heal wounds.
- The matrix is fully synthetic and therefore poses no risk of transmitting diseases or causing immunological reactions.
- **SUPRA SDRM[®] CW** with its unique structure and safe as well as beneficial components is 100% biodegradable. It is completely metabolized to H₂O and CO₂.
- No preparation of **SUPRA SDRM[®] CW** matrix necessary.
- When applied to the wound **SUPRA SDRM[®] CW** adheres well to the wound bed without the need for direct fixation.
- **SUPRA SDRM[®] CW** becomes translucent after the application allowing healthcare professionals to easily assess the wound healing.
- **SUPRA SDRM[®] CW** is available in multiple sizes and can be trimmed to meet the individual patient's needs.

INDICATIONS


SUPRA SDRM[®] CW is indicated for use in the management of:

- Venous ulcers
- Arterial ulcers
- Diabetic wounds
- Mixed ulcers



SUPRA SDRM® CW- APPLICATION GUIDE

- Apply **SUPRA SDRM® CW** and cover with non-adherent mesh dressing; apply outer dressings for desired drainage management to maintain a moist wound healing environment.
- Outer dressings may be changed if soiled.

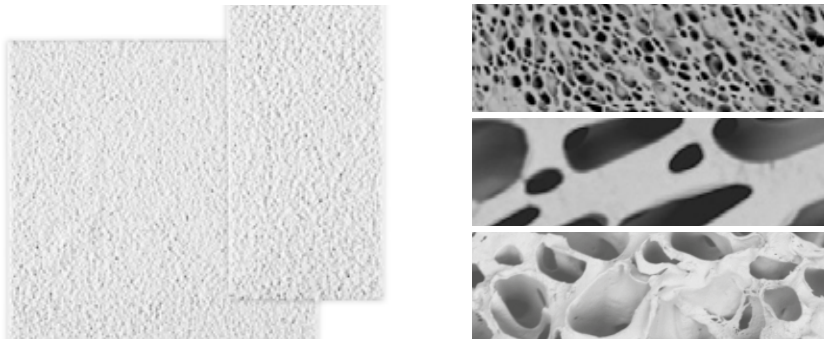

PROPERTIES

Composition	Lacto-capromer
Degradation	Hydrolytical (to CO ₂ and H ₂ O)
Thickness	1.5 - 2.1 mm
Porosity	> 85%
Sterilization	Gamma Sterilization

PACKAGING INFORMATION

Size	Units	Order-No.
Ø 18 mm disc	1	41D018
1 x 1 cm	1	410101
2 x 2 cm	1	410202
5 x 5 cm	1	410505
9 x 9 cm	1	410909
9 x 12 cm	1	410912
18 x 9 cm	1	411809
18 x 18 cm	1	411818

Wound bed preparation



Debride devitalized tissue and refresh the wound with cross hatching as needed. Bring to hemostasis.

SUPRA SDRM® CW - Application



Apply **SUPRA SDRM® CW** to cover wound surface.

Non-adherent separation layer



Apply non-adherent dressing to keep **SUPRA SDRM® CW** in place. Apply outer dressings for desired drainage management. **SUPRA SDRM® CW** may be reapplied as required until wound healing is achieved.

Contact:

PolyMedics Innovations GmbH

Heerweg 15 D
73770 Denkendorf
Phone +49 (0)711 719 500-0
Fax +49 (0)711 719 500-10
Email info@polymedics.com

